




ULTIMA A6

User Manual

Introduction

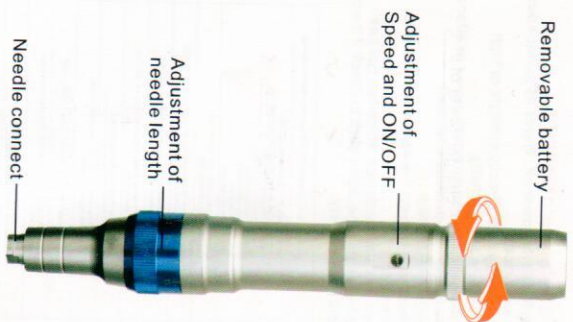

Parts Name



How to Operate

Battery Mode

How to interchange the battery




(1): unscrew the cover of plug (picture 1)
(2): replace the battery (picture 2)

How to charge the battery



(1): put the battery into the charger (picture 3)
(2): plug in DC adaptor to the jack and then connect power supply (picture 4)

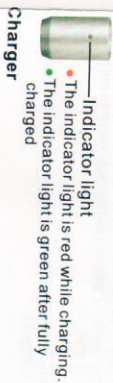

Charger

How to Operate

Adapter Mode

(1) Assemble the cover of plug. (picture 5)
(2) Plug DC adaptor to the jack. (picture 6)
(3) Plug in DC adaptor to AC outlet.



How to install a cartridges

(1) Take out needle cartridges in the sterilized packing.
(2) Fitting round slots 1 into 2 of machine, push needle cartridges horizontally up to the end.
(3) Lock the cartridge by turning it to the counter clockwise.
(4) Adjustment of needle length should be done after turn on the machine.
(5) If you want to remove the needle cartridge, turn it to the clockwise and push out horizontally, fitting round slots (slot 1, slot 2) together.

